



CHEMOCAP PRODUCTS

NON INVASIVE PRODUCTS CLINICALLY PROVEN TO PREVENT CHEMOTHERAPY-INDUCED HAIR LOSS AND NAIL TOXICITY

DIRECTIONS AND INSTRUCTIONS FOR THE USE FOR VASOCAP



Overview

The Vasocap is durable and reusable designed to provide effective treatment for ischemic stroke victims in conjunction with emergency medications.

Each Vasocap is designed and guaranteed to deliver excellent results for a period of 2 years from the date of purchase when used as directed.

Directions For Use

- Seal the Vasocap in a freezer bag and place in a freezer for a minimum of 5 hours prior to use.
- Transport the Vasocaps in a cooler with dry ice in order to maintain the Vasocaps low temperature.
- Use a disposable moist shower cap under the Vasocap to ensure efficient cold transfer to the scalp.
- Insert cotton strips between the skin and the Vasocap to prevent frost bite around the ears.
- Secure the Vasocap with the strap for a snug and comfortable fit.
- The Vasocap retains the required temperature for a period of 30 minutes.
- If necessary a second frozen Vasocap can be used while the patient awaits emergency care.

Contraindications

There are no contraindications associated with the use of the Vasocap.

Cleaning Instructions

- Gently wipe the Vasocap with a damp cloth and a small amount of mild soap.
- Avoid soaking or submerging the Vasocap in water or in any liquid.
- After cleansing allow the Vasocap to air dry.
- Reseal the Vasocap in a freezer plastic bag and replace in the freezer.
- Our objective is to provide excellent service and support to our clients. Feel free to email us with your feedback and suggestions.

2679 Howard Avenue, Windsor, ON, N8X3X2 - Canada [Email: chemocap@chemocap.com](mailto:chemocap@chemocap.com)

Phone: 1-519-971-7777 WWW.CHEMOCAP.COM Fax: 1-519-971-0567